

Table of contents

RARE DISEASE DIAGNOSIS

- Impact of rare disease globally
- The value of genomics for rare disease diagnosis
- Limitations of iterative testing

WHOLE-GENOME SEQUENCING

- 7 The most comprehensive test for rare disease
- 9 WGS offers the highest likelihood of finding a diagnosis for rare genetic disease
- 11 One test to replace many
- 13 Advances in genomic testing are leading to answers faster than ever before
- 16 Understanding a WGS report

CLINICAL MANAGEMENT

- 17 A diagnosis can be life-changing
- 19 Treatment options for genetic diseases
- 21 Patient selection
- WGS for neurological indications
- 24 WGS in clinical practice
- 26 WGS is cost-effective when used as a first-tier test
- 28 Global access to genomic sequencing for rare disease


IMPROVING HUMAN HEALTH BY UNLOCKING THE POWER OF THE GENOME

- 30 Ending the diagnostic odyssey through collaboration and innovation
- 31 A promising future for all

Rare disease diagnosis

Impact of rare disease globally

A rare disease affects only a small group of individuals. Yet, the term small is relative. In the United States, a rare disease is defined as affecting fewer than 200,000 Americans. In Europe, a disease is considered rare when it affects fewer than 1 in 2000 individuals.² A disease with a prevalence of 1 in 2500 counts as rare in Japan.3 With as many as 7000 known rare diseases and many yet to be discovered, these small groups add up.4 Collectively, 2%-6% of the global population (nearly 300 million people) are affected by a rare disease.4-6

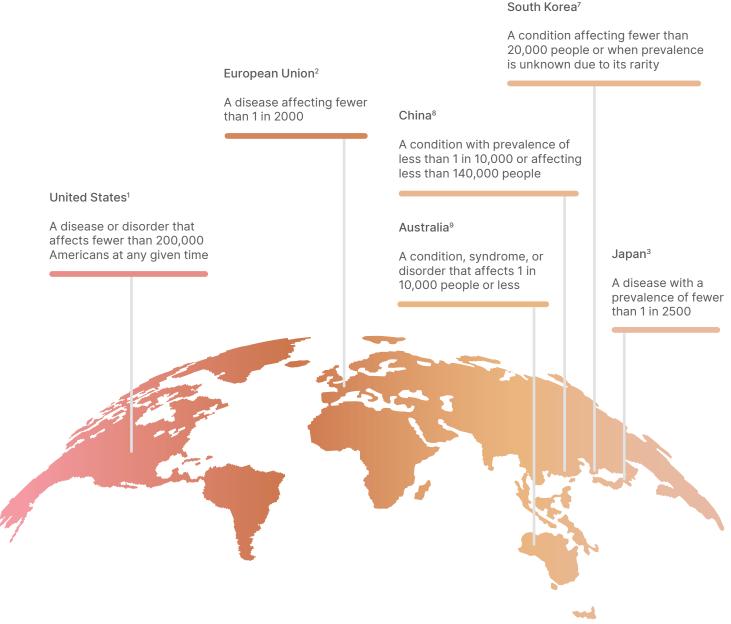



Unfortunately, many individuals affected by rare disease are likely to go undiagnosed, misdiagnosed, and untreated without access to reliable and accurate clinical tools. 10 Children account for approximately 50% of patients affected by rare diseases and 30% of these children will not reach their fifth birthday. 11,12

On average, the long search for a rare disease diagnosis—the "diagnostic odyssey":



May involve multiple tests



Includes up to eight



Takes five to seven years¹³⁻¹⁶



Results in two to three misdiagnoses^{10,14}

Up to

80%

of rare diseases are genetic or have a genetic component.4,5,12,14

Understanding the genomics of rare disease can help doctors pinpoint the cause of undiagnosed disorders and help families avoid years of hospital visits and unnecessary tests. Comprehensive genomic analysis using next-generation sequencing (NGS) increases the potential of uncovering an underlying etiology in patients. 12,17,18 It offers the highest likelihood of rare disease diagnosis^{19,20} and a faster path to ending the diagnostic odyssey.¹⁹

The value of genomics for rare disease diagnosis

Genomics is driving a fundamental shift in rare disease diagnosis, from symptom analysis to molecular etiology assessment. Understanding the biological basis of disease can lead to better care and targeted treatment, with predictable, evidence-based outcomes. Molecular diagnosis in rare disease genomics can provide an opportunity for precision medicine.

Molecular diagnosis of rare disease is a critical step that can benefit patients, their families, physicians, and other care providers. According to the American College of Medical Genetics and Genomics (ACMG), the identification of the genetic etiology of an individual's disease has utility for the patient, their family, and society at large.21

Identifying the genetic cause of an individual's disease can:



Prevent additional unnecessary testing



Lead to the development of new therapies and management strategies



Enable informed family planning



Provide opportunities for psychosocial support via disease support groups^{5,22,23}

Limitations of iterative testing

What you may be missing with current testing applications

There are many ways to investigate the genetic cause of disease, yielding varying results. Traditional methods for genetic analysis are limited in the type of variants they detect and the amount of genome coverage they provide, reducing their potential utility.



Single-gene tests can identify variants or changes in a single gene. Single-gene tests are typically used to confirm or exclude a condition where there is a clear phenotype or family history.



Multigene panels focus on a selection of genes that are typically associated with an indicative phenotype. Multigene panels may not allow for examination of new and emerging targets.



Chromosomal microarrays (CMAs) are designed to detect copy number variants (CNV). CMAs analyze < 0.01% of the genome, missing opportunities to find underlying genetic causes for disease.24



Whole-exome sequencing (WES) sequences the protein coding regions of genes that account for around 2% of the genome, leaving 98% unexplored. WES has limited capability to detect CNVs or structural variants (SV), and fails to detect repeat expansions or paralogs.²⁵⁻³³

Often, these focused methods are chosen as first-line tests based on phenotype. Historical guidance from societies associates specific clinical presentations with certain variant types. For example, because cases of multiple congenital anomalies are commonly associated with aneuploidy and other CNVs, CMAs are frequently used as a first-line test.34 Similarly, multigene targeted NGS panels may be used for patients with conditions, such as epilepsy, that are associated with single nucleotide variants (SNVs) or small insertionsdeletions (indels) in multiple genes.35

However, with many rare diseases the phenotype can be unclear and complex.³⁶ Inconclusive results from initial tests can prompt more genetic tests. This iterative testing places additional burdens on an already stressed healthcare system, requires multiple patient samples, adds complexity to test ordering, and increases the cost and time to answer.

A single comprehensive test that sequences the whole genome can provide more information and be completed more quickly than multiple, iterative tests.³⁷ Whole-genome sequencing (WGS) offers comprehensive genome coverage to maximize testing efficiency and speed to a diagnosis.

QUENCII	JUTIG PANEL VELS JLE EX EX ENOML ES JING CI HR IAL M	OLE EXC GENOM :QUENC OMOSO ROARR' //IGENE WHOLE MEGEN SEQU' IROM CRO/ LTIG	LS WHOLE EGENOM GREAT GUENCI CHROMC MICROA TIGENE	E GENOME	JOMAL MIC JOARRY PT JETIGENT S. WHOL E GENON QUENCIN MOSOMJ ICROARF ULTIGEN S. WHOLL TENOME S. VCING CHI	"GENE PAN WHOLE L ENOMES ING CHR WHOLE L WHOLE L WHOLE L WHOLE L WHOLE L WHOLE L WHOLE CHR WHOLE	UAHINT I NE PANE HOLE E E GENOI ENCING ISOMAL OARBY I NE PANE HOLE E E GENOI ENCING ISOMAL OARBY MULTIGENE PAN WE PANE IS WHOLE FAX	EQUENC CHROM MICROA JIGENE	GEN MAL MICROAR MAL MICROAR MAL MICROAR MAL MICROAR GENT	JOARRY ALTIGENE ANELS W XOME G SEQUENT CHROMO MICROAL MULTIGE PANELS I XOME GL "QUENCIN "OMOSO	MAL MICHOENE NO MULTIGENE NO ME GE IL MICROARF ULTIGENE PA WHOL ME GE QUENCI JUNOSOM MAL MICROARF (MULTIC VE PA	N SEQUEI IROMOS CROARF. ILTIGENE PANELS WI WHOLE EXOME GEN E ME GEN N SEQUEI IIN IROMOS ILTIGENE PANELS WI AL LTIGENE PANELS WI ROMOS LTIGENE PANELS WI	ON SEQUENC CHROMO MAL MIC MULTIG SOME (DI SEQUEN CHRON CHRO CHRON CHRO CHRON CHRON CHRON CHRO CHRO CHRO CHRO CHRO CHRO CHRO CHRO	IN IOSOM/ SG CROARF ROA JLTIGEN ENE I S WHO! DLE EX. GENON VOME S UENCIN G CHR L OSOM/	JARRY MI LITGENE P NELS WI COME GE EQUENC HROMOS IICROAR ULTIGEA WELS W OME GEI UENCING WOSOMA	L MICHON- JUTIGENE P. 'S WHOLE ME GENC JENCING JSOMAL DARRY A WE PANE HOLE E: HE GENO JUENCINI AMOSOM/ L MICROAR' JUTIGEP''	E GENO SE EQUEN G C IOMOSC L I ROARR' L' TGENE L		IN INOMOSOMAL MICROALOS S. L. MICRO-ON MILITARY J. M. L. MICRO-ON MILITARY J. M. L. MICRO-ON MILITARY J. M. MICRO-ON MILITARY J. M. MICRO-ON J. L. MICRO J. M.
.⊿0S0MAL	HROMOSUm. L Microarry M.	: PAN	ULTIGENE PANELS IELS WHOLE EXOM	IE GENI	JENCING CH	ROMOSOMA.	JMAL MICH RRY MULTI	IENE PANE WHOLE EX	IULE EXUME GENUME ENOME SEQUENCING	CHROM	IG CHRUMUS MAL MICROA	AKKY MU ENE PANE		DLE EXOME . NOME SEQUEN.	QUENCING ROMOSON	MUSUMAL I			_ cxome genom.
AOARRY MU	FNE PANELS OLE EXO		ME GENOME SEQU Sequ	ENCIN:	JMOSOMAL MIC MICROARF	ROARRY MUL.	PANELS W ILE EXOME	ENOME SE JENCING C	CING CHROMOSOMAL OSOMAL N		MULTIGENE F Panels whol) LE EXON ENOME S	ING CHRO!	ING CHROMOSOM MICROARA	CROARRY I	SENE PANEL WHOLE EX			CING CHROMOSOMAL MICH DSOMP RRY MULTI
NELS WHO	ENOME				Y MULTIGF	VHOLE EXC	ENOME SE	ROMOSOM	CROARRY		OLE EXOME GE		MOSOMAL'	MULTIGEN	NELS WHO)ME GENOM			ROA/ NE PANEL!
OME GENO.		:OMA			WELS WH	NOME SEC	CING CHR	CROARRY	SENE PANI		OME SECUENCE		ARRY MU		OME GENO	QUENCING		(ICRO/ VULT	
NUENCING CH		IY M			OME GEN	NG CHROM)MAL MICR	ENE PANE	IOLE EXON		G CHRON ISON	MALI ARRYMU A	E PANELS						
PMOSOMAL	L MICRU.	: PAN	IELS WHOLE EXOM	IE GEN IE	SEQUEN	DMAL MICE	RRY MULTI	WHOLE EX	ENOME SEQUENCING		MAL MIC 'ARE	RYML ENEPANE H							
	JLTIGENE PAN.		ME GENOME SEQU		IROMOSC			ENOME SE	CING CHROMOSOMAL										
NAU	IELS WHOLE EXO		SEQUENCING CHR			E PANELS V			OSOMAL MICROARRY				CING CHI						
	"ME GENOME				YMULTIC	HOLE EXON		ROMOSOM	CROARRY				NOSOMAI		NELS WHO				
	"NCING CI		L MIC		INELS WI	NOME SEQ		CROARRY I	SENE PANI		OME SEC		ARRY ML		OME GENO	QUENCIN	MOSOM) LI		
	MAL M				OME GEN(ING CHRON SOMAL MIC)MAL MICR	IENE PANE	IOLE EXON		G CHRON		E PANELS	IL ULITUMI	QUENCING	MOSOMA	ARRY MUL		
HROMOSO ICROARRY	IRY MU PANELS		IELS V Me gi		SEQUENCI ROMOSOMA	RRY MULTI	RRY MULTIA ANELS WHO	WHOLE EX . GENOME S ¹	ENOME SE CING CHR(MAL MIC MULTIGI	1ULTIGENE PANE S WHOLE EXOM	OLE EXOM. OME SEQUE:	¿QUENCIN/	ROMOSON CROARRY	CROARRY BENE PAN	NE PANEL 5 EXOME		ENO). NG CHROM Cing C. Mal Micro/
	FANELS WHOLE EXO		ME GI SEOUENCING CHR		MICROARRY Mu		E EXOME GENOME		OSOMAL MICROARRY		PANELS	ME GENOME S		SUMAL MICROAF	JLTIGENE F	WHOLE	TME SE		DSOMAL MICHOARRY MULTIG
	EXOME GENOM		G CHROMOSOMAL			NELS WHOLE EXU.	ME SEQUENCIN		CROARRY MULTIGENE		OLE EXO	'EQUENCING		ROARRY MULTI	NELS WHO)ME GEN(CING C		CROARRY MULTIGEN'S PANELS
	E SEQUENCIE!		I MICROARRY MII			OME GENOME SEQU	UEIN "POMOSOM		SENE PANELS WHOLE		NME SEN	IMOSOMO		ENE PANE	OME GENO	OHENCIN		IICRO/	THE PANELS WIT FYOME

The most comprehensive test for rare disease

WGS provides the most comprehensive analysis of genomic variants among all clinical genomic testing methods. ^{25,27,29,38} WGS examines the entire genome and has the capability to assess variants in both coding and noncoding regions of the genome. ^{25,27,29,30,33,39,40} Importantly, WGS can detect multiple variant types in a single assay, which improves testing efficiency for rare disease cases of heterogeneous etiologies. ^{25-27,29,30,33,39,40}

Advantages of WGS



Get to a diagnosis faster, with lower costs^{25,41}



Find actionable answers, even when a negative result is returned¹⁷



Enable more personalized care management than other genomic tests⁴²



Obtain a comprehensive view across the genome, including coding and noncoding regions²⁵



Detect a diverse range of variants in a single assav^{25-27,29,30,33,39,40,43}


Variant types detected using WGS compared to standard testing

	Sanger*	Targeted NGS*	PCR*	CMA*	WES*	WGS*
Single nucleotide variants (SNVs) ²⁵		•				
Insertions-deletions (Indels)		•				
Copy number variants (CNVs) ^{25,26}					•	
Repeat expansions ²⁹⁻³²						
Structural variants (SVs) ^{27,28}					1	
Mitochondrial ²⁵		•				
Paralogs ³³					•	
				Limited capa	abilities	Capable

^{*}Variant detection may vary depending on particular laboratory and performance limits of validated variant types. Detection of repeat expansions by PCR is typically limited to single-gene analysis, compared to multigene capabilities for WGS. Improvements to SV variant callers and increased success with long-read whole-genome approaches suggest fully capable SV insights from WGS. NGS = next-generation sequencing, PCR = polymerase chain reaction

WGS captures CNVs with greater resolution than CMA.^{25,40,44,45} WGS also captures some variants in exomes with greater accuracy than WES.^{25,44,46-48} Because WGS provides better coverage and higher yield across the genome, including GC-rich regions, the European Society for Human Genetics (ESHG) guidelines recommend use of WGS, even if only the exome or specific genes are examined bioinformatically.⁴⁹

Many rare disease diagnoses found with WGS would have been missed by WES, including those caused by repeat expansions or mutations in noncoding regions.^{29-32,46,47,50}

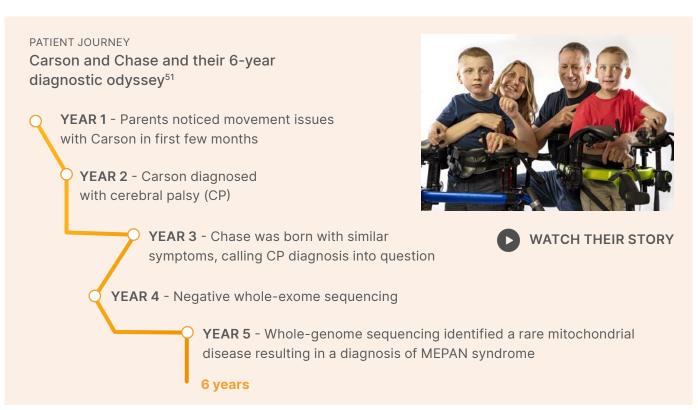

WGS offers the highest likelihood of finding a diagnosis for rare genetic disease

WGS for rare disease has the power to help health care providers diagnose genetic diseases quickly, helping families avoid long diagnostic odysseys. The goal of WGS is to provide a diagnosis with a single test versus multistep iterative testing.

In critically ill infants and in a pediatric outpatient setting, WGS has been shown to have a superior diagnostic yield compared to standard testing and significant evidence of improved clinical management. Combined data from 37 studies comprising 20,068 children found an 8.3× increase in diagnostic yield with WES/WGS compared to CMA.19


Higher diagnostic yield


Clinical performance and utility of WGS has been demonstrated by over 40 peer-reviewed published studies in neonatal or pediatric patients with suspected genetic disease. Together, these reports include more than 9000 patients from around the world. 23,25,32,40-44, 46,48,50,52-83

Highlighted here are eight studies that performed head-to-head comparison of WGS to standard tests.† Four of these studies looked at diagnostic yield for patients in neonatal or pediatric intensive care units (NICU/PICU)42,43,53,59 and four looked at yield in pediatric outpatients.^{23,40,61,66} In both settings, WGS yielded a higher percentage of diagnoses than standard approaches and made a clear impact on clinical management.

[†] Standard tests may include CMA, fluorescence in situ hybridization (FISH), karyotype, targeted gene panels, microarrays, methylation, WES, or other.

WGS has been shown to improve diagnostic yield in both critical care and outpatient settings

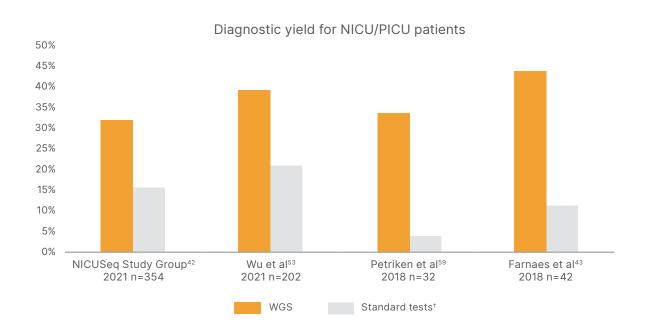

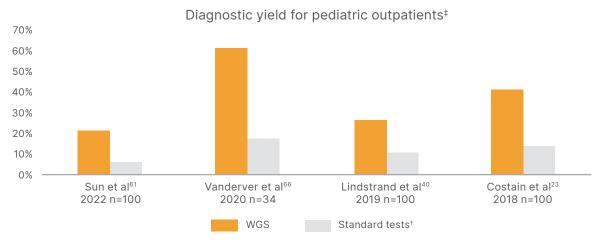


[†] Standard tests may include CMA, FISH, karyotype, targeted gene panels, microarrays, methylation, WES, or other. Cross-trial comparisons cannot be made given different study parameters/design. ‡ Indications may vary across studies.



One test to replace many

WGS may offer the greatest success in finding a diagnosis in rare disease. A single, comprehensive WGS test can provide more information and be completed more quickly than multiple, iterative tests. In addition, WGS data can be stored and reanalyzed as new gene–disease associations are discovered.

In patients with dysmorphism, multiple congenital anomalies, developmental delay, or intellectual disability, the phenotypic presentation often does not suggest a specific genetic condition. In these cases, the standard diagnostic workflow can involve many rounds of targeted genetic assays and/or genome-wide approaches. A workflow that integrates WGS as a first- or second-tier test greatly improves testing efficiency and has the potential to bring a rapid end to the diagnostic odyssey.^{37,86}

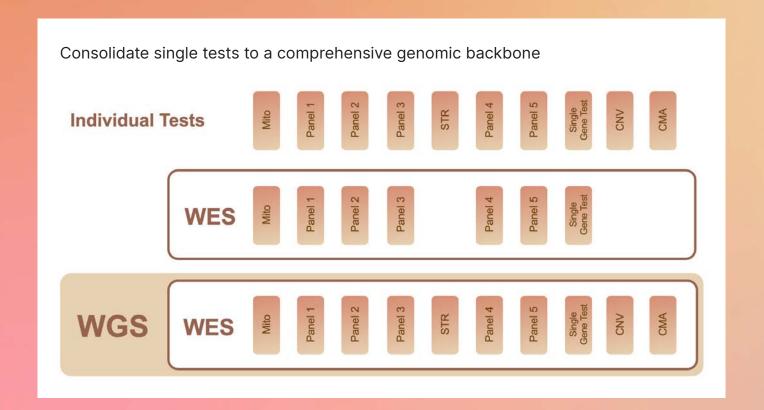

For many rare diseases, faster speed to diagnosis can be critical. For example, infantile encephalopathy is associated with over a thousand genetic diseases, which are difficult to distinguish clinically. Some of these diseases have unique, effective treatments that, when given promptly, can prevent permanent neurologic injury or death.⁸⁴

GENOME SEQUEN.
VCING CHROMOSC

VCROARRY MULTI
GNOSOMAL MICROA

INOME SEC UENCINI
INO

ULTIGENE PANELS
LS WHOLE EXOME (
XOME GENOME SEOL
ME SEQUE NCING CHR
ULTIGENE NELS WHO
MES WHOLE SWHO
MES WHO CHROMOS
HOMOSOMM CROARRY N
MAL MICRO MULTIGENE
HRY MULTIGENE PANELS WHO
PANELS WHOLE EXOME GENOM
MICROARRY MULTIGENE PANE
HRY MULTIGENE PANELS
HRY MULTIGENE PANELS
HRY MULTIC

NCING CH JOMAL MICP
MOSOMAL (JOARRY MU'
MOSOMAL (JOARRY MU'
MICROARRY LTIGENE PP
NOME SEQU JNCING CH'
MAL MICRO
TRY MULTIGENE PP
LS WHOLE EXOL
LE EXOME GENOM.
LING CHRC
JNOME SEQUENCING
SOMAL MI
LE EXOME
GENOME (JOARRE)
LE EXOME
JOARRE

RRY MULTIGENE F
PANELS WHOLE E)
LE EXOME GEOUENCI
ROMOSO MAL MICR
RRY MULTIGENE
PANELS WH
LE EXOME GENO
RESEQUENCI
LINCOR, RRY MI
LIGENE
RALES WH
LE EXOME GEN
RESEQUENCI
LINCOR, RRY MI
LIGENE
RALES WH
LE EXOME GEN
RESEQUENCI
ROMOSON
LE EXOME GEN
RESEQUENCI
ROMOSON
RESEQUENCI
LE EXOME GEN
RESEQUENCI
ROMOSON

QUENCI CHRO
MICRO
MICRO
MICRO
MICRO
MULTI
MULTI
MULTI
MULTI
MOSOM
GENE P
DLE EXO
ELE GENO
ELE GENO
EQUENCI
MOSOM
RO
MOSO

MULTIGENE PANELS WHO
EXOME G
MICROARR
MULTIGENE
EXOME GEV
SEQUENCIN/
ACING CHRO'
AMOSOMAL M'
ROARRY MU'
ANCING CHRO'
CHROMOSOM'

TIGENE PANELS WHOLE EXC WHOLE EXOME GENOME SEC IE GENOME SEQUENCING CH EQUENCIN ENER PANE WHOLE EXC ENCING CHROMOSOMAL M OMAL MICROARTY MULTIGI ROBARTY MULTIGENE PANE TIGENE PAI WHOLE EXC RRY MULTI TIGENE PAI

your diagnostic potential

"In situations where there is not the luxury of waiting, I see it as a moral imperative and an obligation for us to do everything possible in these cases to get to an answer as quickly as possible."

Luca Brunelli, MD, PhD Neonatologist University of Utah Healt



Advances in genomic testing are leading to answers faster than ever before

Studies show that WGS can save years on the time from symptom onset to diagnosis compared to standard genetic testing. 59,76 In a large, randomized controlled trial, the median time to diagnosis in NICU/PICU patients was 13 days with WGS, compared to 107 days with standard testing.59

WGS can provide answers faster than standard testing§

Acutely ill NICU infants:

Time to diagnosis using WGS vs standard genetic tests in the NICU

Rapid WGS = 13 days Standard testing = 107 days



Pediatric patients:

Average time to diagnosis using WGS vs standard genetic tests in pediatric patients

WGS = 1.3 monthsStandard testing = 77 months



[§] Standard tests may include: CMA, FISH, karyotype, targeted gene panels, WES, methylation studies, and gene detection or duplication assays.

PATIENT JOURNEY

Sawyer was on an 8-year diagnostic odyssey before his family found an answer with WGS.85



Rapid and ultra-rapid WGS protocols are further speeding the path to rare genetic disease diagnosis in acutely ill infants and allowing earlier intervention that may improve outcomes. 43,54,57,58,84,86,87



PATIENT JOURNEY Sebastiana's rapid diagnosis with WGS

- Sebastiana, a newborn girl with persistent seizures, was dependent on feeding tubes to survive. Diagnostic workups, including brain MRI, yielded no results.
- Rapid WGS** zeroed in on Sebastiana's exact diagnosis within four days: a novel de novo KCNQ2 mutation associated with Ohtahara syndrome.
- Her medication was changed immediately, her seizures were controlled, and she was discharged home on Christmas day. Since then, Sebastiana has continued to grow and delighted her parents and doctors with her progress.88

^{**}In the NICU and PICU, rapid WGS and ultra-rapid WGS are often used because of their rapid turnaround time given the critical nature of the patients. Ultra-rapid WGS has a median turnaround time of 2.2 days; rapid WGS has a median turnaround time of 3.2 days; 3–14 days turnaround time is routinely achievable.

Understanding a WGS report

Although WGS reports may vary from lab to lab, the content that is included tends to be similar across reports. Here are terminology and guidance on the key information to focus on in a WGS report.

Results

	What may be included	What to look for
+ Positive	Identification of pathogenic or likely pathogenic variant that may fully or partially explain the patient's phenotype	 Whether variant fully or partially explains phenotype Inheritance pattern Supporting evidence used to classify variants
- Negative	No variants identified that explain the patient's phenotype	N/A
NUS	Variant identified that is of uncertain significance in relation to the patient's phenotype	 If a VUS is reported, what supporting evidence was used to classify it as such?
Other	May include secondary, incidental, or other findings	What additional findings were identified and are they relevant to the patient's phenotype or medical management?

Pathogenic variant

Well-established as a cause of the patient's disease

Likely pathogenic variant

Considered a probable cause of the patient's disease

Variant of uncertain significance (VUS)

No or very little evidence to confidently support or rule out pathogenicity

Likely benign variant

Not likely to be the cause of the tested disease

Benign variant

Common polymorphism, not considered to be the cause of the tested disease

Clinical management

A diagnosis can be life-changing

When WGS is implemented early in the diagnostic pathway, it has the potential to offer life-changing options to patients and their families.84

Changes to care my include:







Referral to specialists



Access to precision medicine-based approaches



Avoidance of unnecessary procedures or treatments



Informed reproductive risk counseling for parents and other family members



PATIENT JOURNEY Cocktail treatment for Shubao89

Shubao suffers from hypertonia, which stiffens his muscles making it difficult to move, eat, or sleep. CMA and WES testing and visits to medical specialists failed to find his genetic anomaly. Finally, WGS showed he had lactic acidosis from a mutation in his PDHX gene. Doctors developed a cocktail treatment for Shubao and he showed almost immediate improvement. His muscles relaxed, he slept more, and he started eating.



WATCH THE VIDEO

WGS has been shown to impact clinical management in both critical care and outpatient settings


	Study	Year	Location	Change in clinical management ^{††}
	Suzuki et al ⁵²	2022	Japan	49% New treatment or therapeutic options, specialty referrals, avoidance of invasive testing, and palliative care
	Dimmock et al ⁵⁴	2021	United States	32% Change in surgical procedures, medication, diet, and length of hospital course
	NICUSeq Study Group ⁴²	2021	United States	31% Subspecialty referral, palliative care, medication changes
	Wang et al ⁵⁶	2021	China	43% Therapeutic strategy change, including transplant, diet, or medication change
	Wu et al ⁵³	2021	China	21% Targeted treatments or specialty referrals
NICU/PICU patients	Sanford et al ⁵⁸	2019	United States	76% ^{‡‡} Genome-informed changes in pharmacotherapy and transition to palliative care
	French et al ⁴¹	2019	United Kingdom	70% Modification of treatments and care pathways and/or informing palliative care decisions
	Mestek-Boukhibar et al ⁶⁰	2018	United Kingdom	30% Counseling on prognosis, avoidance of unnecessary investigations, and informed recurrence risk
	Farnaes et al ⁴³	2018	United States	31% Avoidance of invasive test and/or transplant, reducing patient costs by \$800,000–\$2,000,000
	Petrikin et al ⁵⁹	2018	United States	31% Consideration of acute precision intervention in time for critically ill patients
	Sun et al ⁶¹	2022	China	43% Identification of targeted treatments, cessation of unnecessary treatment, and considerations for family planning
	Lee et al ⁷²	2021	Taiwan	23% Immediate changes in treatment strategies after undergoing WGS
Pediatric outpatients	Beuschel et al ⁶⁹	2021	United States	60% Changes to medication, avoidance of additional testing, palliative care
	100,000 Genomes Project ⁴⁶	2021	United Kingdom	25% Immediate implications for clinical decision making
	Scocchia et al ⁶⁷	2019	Mexico	49% Referrals to specialists, avoidance of invasive muscle biopsies, additional clinical investigations, genetic counseling, and palliative care

^{††} Percentage is based on those with a positive WGS result.

^{‡‡ 24%} for rapid WGS during PICU stay, 82% for rapid WGS after discharge.

Treatment options for genetic diseases

Specific and effective treatments are available for many rare genetic diseases. Orphan drugs indicated for a small number of patients are just one example of rare disease treatments.90 Other options may be special diets or supplements to treat symptoms related to disease pathology. WGS-enabled diagnoses can lead to a growing number of effective interventions in acute care settings.

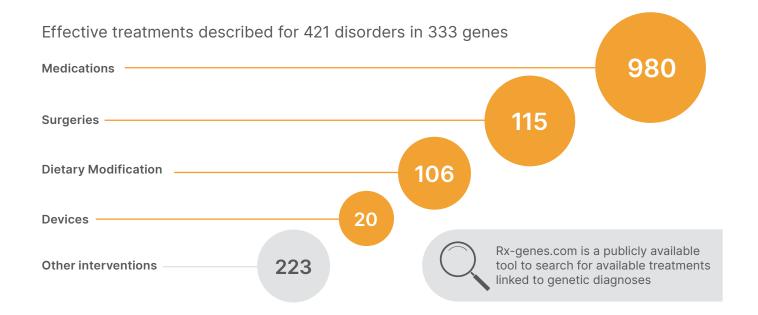



Support for precision medicine

There are tools to help health care providers easily search for available treatments that are linked to a genetic diagnosis. The following online resources are continuously updated to synthesize the published treatment literature and provide guidance and practical information for front-line physicians.86

- Rx-genes.com is an online compendium of over 633 treatable genetic disorders⁹¹
- Genome-to-Treatment (GTRx[™]) is a virtual automated system for immediate clinical management of childhood genetic diseases90



PATIENT JOURNEY Gene therapy trial for Fitz⁹³

As a newborn, Fitz was diagnosed with severe combined immunodeficiency (SCID). Rapid WGS identified his specific mutation in under three days and matched him with a gene therapy clinical trial. The treatment succeeded in creating a functioning immune system for Fitz and he is growing up healthy.



WATCH THE VIDEO

Patient selection

WGS addresses unmet needs among specific patient groups

WGS is especially helpful to provide faster answers for patients with immature phenotypes or those with heterogeneous symptoms.94 The purpose of using WGS is to avoid the anchoring bias of a phenotype-based approach to diagnosis.

WGS may be considered clinically useful when: 12,49,73,95

- The patient's phenotype does not clearly identify a specific disease with an established single-gene or multigene panel, or the patient has phenotypic characteristics outside of, or in addition to, what has been established for the disease
- A definitive diagnosis cannot be made based on standard clinical work up
- A definitive diagnosis will have clinical utility (improvement in net health outcomes); for example:
 - 1. establishing the diagnosis by genetic testing will end the clinical workup for other disorders
 - 2. a definitive diagnosis will lead to changes in clinical management or changes in surveillance
 - 3. the diagnosis leads to changes in reproductive decisionmaking for parents

Specifically, WGS may be indicated for pediatric patients who have multiple congenital anomalies, early onset epileptic encephalopathies, moderate-to-severe intellectual disability, global developmental delay, or other phenotypes that do not clearly point to any specific disease. 12,49,73,95



WGS for neurological indications

WGS demonstrates high diagnostic yield for multiple neurological indications, including developmental delay, intellectual disability, epilepsy, and neuromuscular conditions. A wide range of studies have established WGS diagnostic yield for neurological conditions to range from 25% to almost 60%. 36,40,48,61,62,66,72,74,75




a. Studies combined pediatric and adult cohorts.

WGS is a highly effective first-tier test for unexplained intellectual disability or developmental delay (ID/DD)^{55,76,97,98}

Intellectual disability affects around 1%–3% of the population with around three per thousand being of the severe form. Severe intellectual disability often has a genetic etiology and WGS has proven an effective diagnostic first-tier test for these patients, with higher diagnostic yield compared to traditional testing methods.

Emerging evidence for WGS as a first-tier test

Improved diagnostic yield for intellectual disability⁷¹

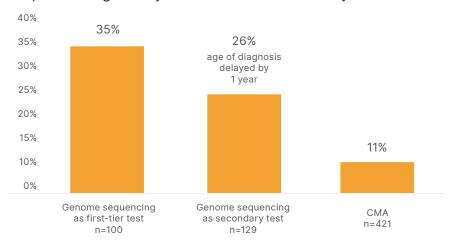


PATIENT JOURNEY Sophia's 10-year diagnostic odyssey99

The long search for a diagnosis was so fatiguing for Sophia and her family that they almost gave up. Learn how they found not only an answer, but a community.





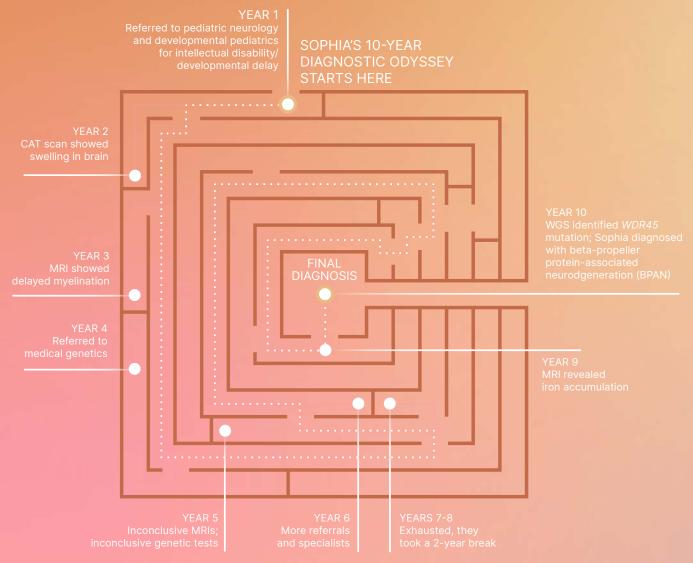

"Clinical WGS has turned out to be a true game changer in the rare disease Space." Stranneheim et al, 202163

WGS in clinical practice

The promise of WGS to improve diagnosis and management of rare disease is being validated in clinical practice and recognized by societies. ACMG and ESHG guidelines advocate genome sequencing when it can improve testing quality, efficiency, or diagnostic yield. 49,100 With inclusion in national healthcare systems 46,63 and increasing evidence of economic value when used as a first-tier test, 54,87,101 WGS appears to be on the path toward standard of care.

The Genomic Medicine Center Karolinska-Rare Diseases (GMCK-RD) in Sweden demonstrated a successful approach to integrating WGS into the management of rare disease in a clinical setting. Out of 3219 rare disease patients over four years, 40% (1285/3219) received a molecular diagnosis with WGS. Variants in 754 different genes were reported. The medical center stated that clinical WGS has turned out to be a true game changer in the rare disease space. 63

Society endorsements

In 2021, ACMG released guidance recommending the use of WES or WGS as first- or second-tier tests in patients with one or more congenital anomalies prior to one year of age or intellectual disabilities/developmental delay prior to eighteen years of age.¹⁷ The ACMG guideline states that genome-wide sequencing leads to:

- Increased diagnostic yield in rare disease
- Awareness of broad spectrum of genetic variants
- Improved patient outcomes
- · Expanded treatment and management
- Access to support networks for patients and families¹⁷

Medical societies across the world also recommend genomewide sequencing as a first- or second-tier diagnostic test for rare disease.17,49,102,103

<u> </u>				
COOLOT	/ ctatamanta an	donomo wido cor	NUMBER OF TAKE	dicasca disappeid
SOCIETY	v statements on	DEHOTIE-WIDE SEC	nuencino ioi iare	disease diagnosis

	ACMG ¹⁷ American College of Medical Genetics and Genomics	CMDA ¹⁰² Chinese Medical Doctor Association, Medical Genetics Branch	RACP ¹⁰³ Royal Australasian College of Physicians, Paediatric and Child Health Division	ESHG ⁴⁹ European Society of Human Genetics
Date of latest publication	July 2021	June 2019	February 2021	May 2022
Sequencing type	WES/WGS	WGS	WES/WGS	WGS
Eligible patients	Patients with one or more congenital anomalies prior to one year of age OR with intellectual disability with onset prior to age 18	Non-specific phenotype associated with intellectual disability and/or developmental delay; multiple congenital anomalies Clear clinical diagnosis associated with high level of genetic heterogeneity Previously negative WES or CMA	Any child < 10 years with: facial dysmorphism AND ≥ 1 congenital structural anomaly; OR global developmental delay/ intellectual disability (moderate to severe); Test must be requested by clinical geneticist OR pediatrician following consultation with clinical geneticist	It is recommended to introduce WGS analysis in a diagnostic setting when it is a relevant improvement on quality, efficiency and/or diagnostic yield

ESHG recommendations for WGS in diagnostics for rare diseases

Guidelines released by ESHG in 2022 recommend WGS analysis in diagnostic settings when it is a relevant improvement on quality, efficiency, and/or diagnostic yield.⁴⁹ The ESHG acknowledges that WGS offers a "potential overall benefit for the patient" and that when used as a first-tier test, "the extra sequencing costs may be compensated by the fact that fewer additional tests will be required."49

The ESHG guidelines also outline best practices for when clinicians are ordering WGS testing:

- Describing the patient phenotype (including negative criteria) in standardized terms and data formats is necessary for interpretation of variants
- Clinicians should provide genetic counseling—conducted by a qualified clinical expert—and obtain informed consent prior to clinical WGS
- Requests for reanalysis of WGS data should be triggered by the referring physician, with patient consent

"WES and gene panels have been commonly used in a diagnostic setting for many years. WGS can be recommended based on increased quality For instance, WGS allows the detection of SNVs and CNVs outside the exome, and provides a better



WGS is cost-effective when used as a first-tier test

The Center for the Evaluation of Value and Risk in Health (CEVR) at Tufts Medical Center in Massachusetts examined the cost-effectiveness of exome and genome sequencing for infants and children with rare and undiagnosed conditions.¹⁰¹ This rigorous study compared standard-of-care genetic testing to seven possible diagnostic pathways, including first-tier WES, first-tier WGS, and combinations of WES and/or WGS after standard care. Cost-effectiveness analysis compared cost and clinical and quality-of-life impact of different modalities. Firsttier WGS was considered the most cost-effective approach, producing the lowest cost per diagnosis and potentially the lowest cost per quality-adjusted years of life. 101 All other strategies led to the same or fewer diagnoses at a higher cost per diagnosis. CEVR found that standard-of-care testing before WES or WGS increases costs without improving outcomes.¹⁰¹

A similar study in Australia evaluated the cost associated with providing rapid genomic results in pediatric critical care vs reduction in health care costs.87 This prospective study analyzed real-world hospital data for 40 critically ill infants with suspected genetic disease and different genetic testing approaches. Early rapid WGS was the most cost-effective strategy with an estimated savings of \$800,000 to \$2 million per 100 patients tested. The study authors found that reducing the time to diagnosis led to significant economic and personal benefits, and they recommended changing clinical practice to expedite WGS test initiation.87

Continuing to establish the cost-effectiveness of WGS in clinical practice is needed across the globe. The following special projects were designed to evaluate the clinical and economic effects of rapid WGS for critically ill children. These pioneering initiatives are generating evidence of clinical utility and economic benefits that will make it easier for other institutions to follow.

Project Baby Bear⁵⁴

Rapid WGS for 184 newborns in NICUs in California

Saved an average of \$12K to \$15K per child's genome sequenced, compared with the \$9K per child cost of the procedure, for around \$1.2 million net savings

Project Baby Manatee¹⁰⁵

Rapid WGS for 50 critically ill babies and children in Florida

Reported a return on investment of \$2.88 million

Baby Bambi¹¹⁰

Pilot study to offer rapid WGS to NICU patients across hospitals in Israel

- Evaluating workflow and feasibility of implementing WGS-based diagnostics into routine care
- · Clinical utility research is ongoing

Baby Lion study^{108,109}

Rapid WGS for 100 children in NICU/PICU and parents (trios) in Germany

Ongoing study to demonstrate the positive impact of earlier diagnosis and targeted treatment


Project Baby Deer^{106,107}

Five hospitals across Michigan offering rapid WGS to newborns and children up to 18 years

- Decreased hospital stays for NICU patients, which led Michigan to be the first state to mandate Medicare coverage of rapid WGS
- Clinical utility research is ongoing

Global access to genomic sequencing for rare disease

Improving care for rare disease is a global mission. Nations and health systems across Europe, North America, and East Asia are increasing coverage for WGS and WES as they recognize the value of genome sequencing for rare disease diagnosis. Special initiatives like the 100,000 genomes projects in the United Kingdom and Singapore are committed to expand knowledge about the role of genomics in rare disease. 46,48,112

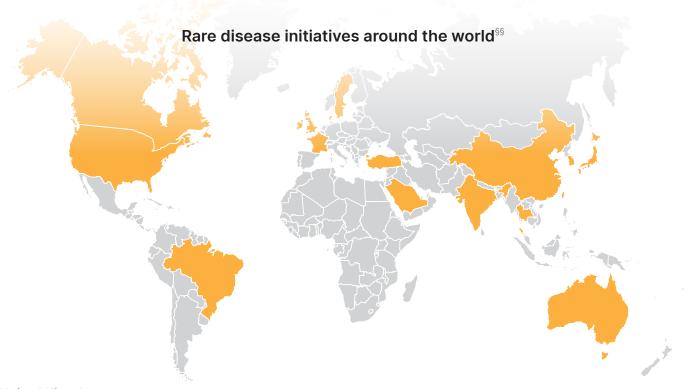

United Kingdom

The UK has set an example for the world with the 100,000 Genomes Project to study the role of genomics in rare disease and cancer. 46,48 The Wales Infants and Children's Genome Service (WINGS) and the National Health Service (NHS) in Wales were the first to offer rapid WGS as a diagnostic service for acutely unwell children. Since 2019, the NHS in England covers WGS as a routine diagnostic test for over 30 different clinical presentations. For patients with specific disorders, such as intellectual disability, WGS is a first-line test in the NHS.

Canada

In 2020, the Ontario Health Technology Assessment (HTA) Series concluded that genome-wide sequencing (including WES and WGS) has a higher diagnostic yield than standard genetic testing (CMA, targeted singlegene tests, or gene panels) and can prompt changes to some medications or treatments and referrals to specialists. The HTA also stated that genome-wide sequencing could be a cost-effective strategy when used after standard testing to diagnose people with unexplained developmental disabilities or multiple congenital anomalies and could also lead to cost savings when used earlier in the diagnostic pathway. Patients and families consistently noted a benefit from seeking a diagnosis through genetic testing.¹⁸

United States

The Blue Cross Blue Shield Association (BCBSA) technology assessment was positive for rapid WGS in the NICU/PICU. Based on critical review of peer-reviewed, published evidence, the BCBSA technical assessment body determined that WGS results in a "meaningful improvement in the net health outcome." 115 Since 2020, Blue Shield of California has covered rapid WGS for sick children up to age 18. Medicaid programs in Michigan and California were the first to cover rapid WGS for critically ill infants up to age one.

Japan

From 2015 to 2021, the Initiative on Rare and Undiagnosed Disease (IRUD) program led to final diagnoses in 2247 out of 5136 pedigrees (43%) and identified over a thousand novel pathogenic variants in 657 known genes.¹¹⁶ The 100K national WGS project is ongoing. About 2500 cases of analysis are planned for the 2022 fiscal year for intractable disease, which covers monogenetic, multifactorial, and difficult-todiagnose diseases.

Korea

The second pilot in National Project of Bio Big Data project will analyze the genetic makeup of 12,500 donated DNA samples from Korean patients living with a rare disease. The effort will work towards establishing a national digital library on health and genome data by 2028.¹¹⁷

China

In 2019, China launched the Newborn Genome Project to conduct rapid WGS for 200 NICU patients and compare diagnosis rates vs traditional detection methods. The project is also assessing diagnosis time, prognosis improvement, and hospitalization turnaround time to provide feasible recommendations for the application of rapid WGS in the NICU.118

Singapore

In 2022, Singapore announced the Precision Health Research Singapore (PRECISE) program to sequence 100,000 genomes of Singaporean participants and gather deep insights into key genetic, social, environmental, and other factors associated with disease.¹¹²


Australia

In 2019, the Medical Services Advisory Committee (MSAC) supported the inclusion of trio testing with WES or WGS for affected patients with dysmorphic facial appearance and one or more major structural congenital anomalies; or intellectual disability or global developmental delay of at least moderate severity to be determined by a specialist pediatrician.¹¹⁹

Germany

In 2021, Germany began reimbursing WGS/WES with no prior authorization requirement at the national level. In addition, major German health insurance companies cover WES and WGS at a higher reimbursement level to allow for testing of patients and their parents (trios).

Other countries that cover WES and WGS for patients with undiagnosed, suspected genetic diseases include Switzerland, Denmark, and Sweden. Several countries, including Belgium, France, the Netherlands, Israel, and Spain are actively pursuing integration of WGS into clinical care for certain indications and/or launching genomic initiatives that would include WGS for rare and undiagnosed disease.

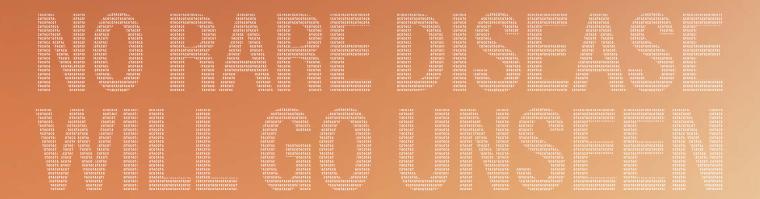

Improving human health by unlocking the power of the genome

Ending the diagnostic odyssey through collaboration and innovation

Illumina strives to make genomics available to the many, not the few. As part of that commitment, Illumina sponsors several collaborations and programs to increase adoption and access to WGS for rare disease diagnosis.

Supporting the journey to the implementation of WGS

Illumina is creating a network of sites across Europe and the Middle East to connect and collaborate as they work to implement genomic sequencing in the intensive care setting. The working group includes 45 participants from 22 institutions across Belgium, Czech Republic, France, Germany, Israel, Italy, the Netherlands, Norway, Saudi Arabia, Spain, Sweden, UAE-Abu Dhabi, UAE-Dubai, and the United Kingdom. Participants share experiences, enabling discussion on technical aspects, resources needed, and clinical considerations such as referral criteria, results reports, and patient follow-up.

Dr Ahmad Abou Tayoun (pictured on the cover) is director of the Genomics Centre at Al Jalila Children's Specialty Hospital in Dubai and a member of the working group. Dr Abou Tayoun and colleagues published a study on rapid WGS for critically ill infants from genetically underrepresented populations. Trio WGS analysis was completed within 37 hours, which provided fast and precise diagnostic findings in three out of five patients and aided in devising better management plans for them in the intensive care setting. For example, one patient was diagnosed with Pallister-Killian syndrome. Another was found to have pathogenic variants in the *LIPA* gene and became a candidate for an FDA-approved enzyme replacement therapy. Even the data for the two infants without a diagnosis could present a research opportunity for gene discovery, which may lead to a diagnosis in the future.

Regarding the clinical utility of rapid WGS, Dr Abou Tayoun says, "Although it looks slightly more expensive than traditional testing, it has its value. It's quick, and there's no unnecessary diagnostic workup or inefficient treatment" 121

The Medical Genome Initiative

The Medical Genome Initiative is a consortium of nine equal participating member institutions with the joint mission to expand access to high-quality WGS for rare disease diagnosis. These thought-leader institutions coordinate projects to develop and publish common laboratory and clinical best practices.¹²² The Medical Genome Initiative also seeks to align the clinical research community on endpoints for measuring WGS clinical and economic utility. Illumina participates as a scientific contributor, administrator, and sponsor.

















Illumina iHope mogram and iHope Genetic Health program

The iHope program is a philanthropic initiative launched by Illumina to make WGS accessible to underserved families with children facing rare and undiagnosed genetic diseases. This program collaborates directly with pediatric health care providers, organizations, and academic institutions to identify patients appropriate for testing. iHope Genetic Health, a program governed, managed, and operated by the nonprofit Genetic Alliance, expands the Illumina iHope program to make WGS broadly accessible to low- and middle-income communities around the world, with more than one-third of funds being allocated to patients in Africa. 123,124

A promising future for all

Whole-genome sequencing has great potential to deliver precise molecular diagnoses and alter future medical management in many patient populations, including those with rare and undiagnosed genetic disease. WGS can provide pharmacogenomic data to inform individualized drug selection and/or dosing adjustment.¹²⁵ As use of WGS grows and that data is shared, more genotype-phenotype associations are being established and information of the pathogenicity of variants will become more certain, and reanalysis of genomes will yield new diagnoses. 49,126

WGS is already in use and showing positive results in critical care units across the globe^{41,42,52,53,56-60,76,70-128} and is recommended by the ACMG and ESHG.^{17,49} With its improved diagnostic performance and faster time to answer, WGS holds the promise of helping patients and their families end an odyssey of inconclusive testing and unnecessary medical interventions—or prevent one altogether—and focus on care management.

- (Learn from others using whole-genome sequencing for rare disease diagnostics
- (>) Learn more about rare disease genomics

References

- Genetics and Rare Disease Information Center. National Center for Advancing Translation Sciences. What is a rare disease? rarediseases.info.nih.gov/about. Accessed January 6, 2023.
- 2. Orphanet. About Rare Diseases. orpha.net/consor/cgi-bin/Education_AboutRareDiseases.php?Ing=EN. Updated October 25, 2012. Accessed January 6, 2023.
- 3. Hayashi S, Umeda T. **35** years of Japanese policy on rare diseases. *Lancet*. 2008;372(9642):889-890. doi:10.1016/S0140-6736(08)61393-8
- 4. Nguengang Wakap S, Lambert DM, Olry A, et al. Estimating cumulative point prevalence of rare diseases: analysis of the Orphanet database. Eur J Hum Genet. 2020;28(2):165-173. doi:10.1038/s41431-019-0508-0
- 5. Ferreira CR. The burden of rare diseases. Am J Med Genet A. 2019;179(6):885-892. doi:10.1002/ajmg.a.61124
- 6. Walker CE, Mahede T, Davis G, et al. The collective impact of rare diseases in Western Australia: an estimate using a population-based cohort. *Genet Med.* 2017;19(5):546-552. doi:10.1038/gim.2016.143
- 7. Kim SY, Lim BC, Lee JS, et al. The Korean undiagnosed diseases program: lessons from a one-year pilot project. Orphanet J Rare Dis. 2019;14(1):68. doi:10.1186/s13023-019-1041-5
- 8. Cui Y, Han J. Defining rare diseases in China. Intractable Rare Dis Res. 2017;6(2):148-149. doi:10.5582/irdr.2017.01009
- 9. Australian Department of Health: Therapeutic Goods Administration. Orphan drugs program. tga.gov.au/sites/default/files/consultation-orphan-drugs-program.pdf. Updated May 2015. Accessed January 6, 2023.
- 10. Bogart K, Hemmesch A, Barnes E, et al. Healthcare access, satisfaction, and health-related quality of life among children and adults with rare diseases. *Orphanet J Rare Dis.* 2022;17(1):196. doi:10.1186/s13023-022-02343-4
- 11. Eurodis Rare Diseases Europe. What is a rare disease? eurordis.org/information-support/what-is-a-rare-disease/. Accessed January 6, 2023.
- 12. Bick D, Jones M, Taylor SL, Taft RJ, Belmont J. Case for genome sequencing in infants and children with rare, undiagnosed or genetic diseases. *J Med Genet*. 2019;56(12):783-791. doi:10.1136/jmedgenet-2019-106111
- 13. Global Commission on Rare Disease. Global commission to end the diagnostic odyssey for children with a rare disease. globalrarediseasecommission.com/. Accessed January 6, 2023.
- 14. Shire. Rare Disease Impact Report: Insights from patients and the medical community. Global Genes website. globalgenes.org/wp-content/uploads/2013/04/ShireReport-1.pdf. Published 2013. Accessed January 6, 2023.
- 15. Global Genes. Allies in Rare Disease. globalgenes.org/. Accessed January 6, 2023.
- 16. Posada de la Paz, Taruscio, Groft. *Rare diseases epidemiology: Update and overview.* 2nd edition. Chapter 2. Springer 2017. Cham, Switzerland. Available from: link.springer.com/book/10.1007/978-3-319-67144-4
- 17. Manickam K, McClain MR, Demmer LA, et al. Exome and genome sequencing for pediatric patients with congenital anomalies or intellectual disability: an evidence-based clinical guideline of the American College of Medical Genetics and Genomics (ACMG). *Genet Med.* 2021;23(11):2029-2037. doi:10.1038/s41436-021-01242-6
- 18. Ontario Health (Quality). Genome-Wide Sequencing for Unexplained Developmental Disabilities or Multiple Congenital Anomalies: A Health Technology Assessment. *Ont Health Technol Assess Ser.* 2020;20(11):1-178.
- Clark MM, Stark Z, Farnaes L, et al. Meta-analysis of the diagnostic and clinical utility of genome and exome sequencing and chromosomal microarray in children with suspected genetic diseases. NPJ Genom Med. 2018;3:16. doi:10.1038/s41525-018-0053-8
- 20. Vissers LE, Gilissen C, Veltman JA. Genetic studies in intellectual disability and related disorders. *Nat Rev Genet*. 2016;17(1):9-18. doi:10.1038/nrg3999
- 21. ACMG Board of Directors. Clinical utility of genetic and genomic services: a position statement of the American College of Medical Genetics and Genomics. *Genet Med.* 2015;17(6):505-507. doi:10.1038/gim.2015.41
- 22. Hayeems RZ, Bhawra J, Tsiplova K, et al. Care and cost consequences of pediatric whole genome sequencing compared to chromosome microarray. *Eur J Hum Genet*. 2017;25(12):1303-1312. doi:10.1038/s41431-017-0020-3
- 23. Costain G, Jobling R, Walker S, et al. Periodic reanalysis of whole-genome sequencing data enhances the diagnostic advantage over standard clinical genetic testing. Eur J Hum Genet. 2018;26(5):740-744. doi:10.1038/s41431-018-0114-6
- 24. Illumina. Data on file. March 2022.
- 25. Lionel AC, Costain G, Monfared N, et al. Improved diagnostic yield compared with targeted gene sequencing panels suggests a role for whole-genome sequencing as a first-tier genetic test. *Genet Med.* 2018;20(4):435-443. doi:10.1038/gim.2017.119
- 26. Gross AM, Ajay SS, Rajan V, et al. Copy-number variants in clinical genome sequencing: deployment and interpretation for rare and undiagnosed disease. *Genet Med.* 2019;21(5):1121-1130. doi:10.1038/s41436-018-0295-y
- 27. Chen X, Schulz-Trieglaff O, Shaw R, et al. Manta: rapid detection of structural variants and indels for germline and cancer sequencing applications. *Bioinformatics*, 2016;32(8):1220-1222. doi:10.1093/bioinformatics/btv710
- 28. Carss KJ, Arno G, Erwood M, et al. Comprehensive Rare Variant Analysis via Whole-Genome Sequencing to Determine the Molecular Pathology of Inherited Retinal Disease. *Am J Hum Genet*. 2017;100(1):75-90. doi:10.1016/j.ajhg.2016.12.003
- 29. Dolzhenko E, van Vugt JJFA, Shaw RJ, et al. Detection of long repeat expansions from PCR-free whole-genome sequence data. *Genome Res.* 2017;27(11):1895-1903. doi:10.1101/gr.225672.117
- 30. Alfares A, Aloraini T, Subaie LA, et al. Whole-genome sequencing offers additional but limited clinical utility compared with reanalysis of whole-exome sequencing. *Genet Med.* 2018;20(11):1328-1333. doi:10.1038/gim.2018.41
- 31. van Kuilenburg ABP, Tarailo-Graovac M, Richmond PA, et al. Glutaminase Deficiency Caused by Short Tandem Repeat Expansion in GLS. N Engl J Med. 2019;380(15):1433-1441. doi:10.1056/NEJMoa1806627

- 32. Ibañez K, Polke J, Hagelstrom RT, et al. Whole genome sequencing for the diagnosis of neurological repeat expansion disorders in the UK: a retrospective diagnostic accuracy and prospective clinical validation study. *Lancet Neurol.* 2022;21(3):234-245. doi:10.1016/S1474-4422(21)00462-2
- 33. Chen X, Sanchis-Juan A, French CE, et al. Spinal muscular atrophy diagnosis and carrier screening from genome sequencing data. *Genet Med.* 2020;22(5):945-953. doi:10.1038/s41436-020-0754-0
- 34. Miller DT, Adam MP, Aradhya S, et al. Consensus statement: chromosomal microarray is a first-tier clinical diagnostic test for individuals with developmental disabilities or congenital anomalies. *Am J Hum Genet*. 2010;86(5):749-764. doi:10.1016/j. aihq.2010.04.006
- 35. Wright CF, FitzPatrick DR, Firth HV. Paediatric genomics: diagnosing rare disease in children. *Nat Rev Genet*. 2018;19(5):253-268. doi:10.1038/nrg.2017.116
- 36. Palmer EE, Sachdev R, Macintosh R, et al. Diagnostic Yield of Whole Genome Sequencing After Nondiagnostic Exome Sequencing or Gene Panel in Developmental and Epileptic Encephalopathies. *Neurology*. 2021;96(13):e1770-e1782. doi:10.1212/WNL.000000000011655
- 37. Sun F, Oristaglio J, Levy SE, et al. *Genetic Testing for Developmental Disabilities, Intellectual Disability, and Autism Spectrum Disorder.* Rockville (MD): Agency for Healthcare Research and Quality (US); 2015 Jun. (Technical Briefs, No. 23.) Available from: ncbi.nlm.nih.qov/books/NBK304462/
- 38. Meienberg J, Bruggmann R, Oexle K, Matyas G. Clinical sequencing: is WGS the better WES? *Hum Genet*. 2016;135(3):359-362. doi:10.1007/s00439-015-1631-9
- 39. Sanghvi RV, Buhay CJ, Powell BC, et al. Characterizing reduced coverage regions through comparison of exome and genome sequencing data across 10 centers. *Genet Med.* 2018;20(8):855-866. doi:10.1038/gim.2017.192
- 40. Lindstrand A, Eisfeldt J, Pettersson M, et al. From cytogenetics to cytogenomics: whole-genome sequencing as a first-line test comprehensively captures the diverse spectrum of disease-causing genetic variation underlying intellectual disability. *Genome Med.* 2019;11(1):68. doi:10.1186/s13073-019-0675-1
- 41. French CE, Delon I, Dolling H, et al. Whole genome sequencing reveals that genetic conditions are frequent in intensively ill children. Intensive Care Med. 2019;45(5):627-636. doi:10.1007/s00134-019-05552-x
- 42. NICUSeq Study Group, Krantz ID, Medne L, et al. Effect of Whole-Genome Sequencing on the Clinical Management of Acutely III Infants With Suspected Genetic Disease: A Randomized Clinical Trial. *JAMA Pediatr.* 2021;175(12):1218-1226. doi:10.1001/iamapediatrics.2021.3496
- 43. Farnaes L, Hildreth A, Sweeney NM, et al. Rapid whole-genome sequencing decreases infant morbidity and cost of hospitalization. NPJ Genom Med. 2018;3:10. doi:10.1038/s41525-018-0049-4
- 44. Stavropoulos DJ, Merico D, Jobling R, et al. Whole Genome Sequencing Expands Diagnostic Utility and Improves Clinical Management in Pediatric Medicine. NPJ Genom Med. 2016;1:15012-. doi:10.1038/npjgenmed.2015.12
- 45. Trost B, Walker S, Wang Z, et al. A Comprehensive Workflow for Read Depth-Based Identification of Copy-Number Variation from Whole-Genome Sequence Data. *Am J Hum Genet*. 2018;102(1):142-155. doi:10.1016/j.ajhg.2017.12.007
- 46. 100,000 Genomes Project Pilot Investigators, Smedley D, Smith KR, et al. 100,000 Genomes Pilot on Rare-Disease Diagnosis in Health Care Preliminary Report. N Engl J Med. 2021;385(20):1868-1880. doi:10.1056/NEJMoa2035790
- 47. Belkadi A, Bolze A, Itan Y, et al. Whole-genome sequencing is more powerful than whole-exome sequencing for detecting exome variants. *Proc Natl Acad Sci U S A*. 2015;112(17):5473-5478. doi:10.1073/pnas.1418631112
- 48. Turro E, Astle WJ, Megy K, et al. Whole-genome sequencing of patients with rare diseases in a national health system. *Nature*. 2020;583(7814):96-102. doi:10.1038/s41586-020-2434-2
- 49. Souche E, Beltran S, Brosens E, et al. Recommendations for whole genome sequencing in diagnostics for rare diseases. Eur J Hum Genet. 2022;10.1038/s41431-022-01113-x. doi:10.1038/s41431-022-01113-x
- 50. Splinter K, Adams DR, Bacino CA, et al. Effect of Genetic Diagnosis on Patients with Previously Undiagnosed Disease. N Engl J Med. 2018;379(22):2131-2139. doi:10.1056/NEJMoa1714458
- 51. Miller D. The diagnostic odyssey: our family's story. Am J Hum Genet. 2021;108(2):217-218. doi:10.1016/j.ajhg.2021.01.003
- 52. Suzuki H, Nozaki M, Yoshihashi H, et al. Genome Analysis in Sick Neonates and Infants: High-yield Phenotypes and Contribution of Small Copy Number Variations. *J Pediatr.* 2022;244:38-48.e1. doi:10.1016/j.jpeds.2022.01.033
- 53. Wu, B, Kang W, Wang Y, et al. Application of Full-Spectrum Rapid Clinical Genome Sequencing Improves Diagnostic Rate and Clinical Outcomes in Critically III Infants in the China Neonatal Genomes Project. *Crit Care Med.* 2021;49(10):1674-1683. doi:10.1097/CCM.00000000000005052
- 54. Dimmock D, Caylor S, Waldman B, et al. Project Baby Bear: Rapid precision care incorporating rWGS in 5 California children's hospitals demonstrates improved clinical outcomes and reduced costs of care. *Am J Hum Genet*. 2021;108(7):1231-1238. doi:10.1016/j.ajhg.2021.05.008
- 55. Bowling KM, Thompson ML, Finnila CR, et al. Genome sequencing as a first-line diagnostic test for hospitalized infants. *Genet Med.* 2022;24(4):851-861. doi:10.1016/j.gim.2021.11.020
- 56. Wang H, Lu Y, Dong X, et al. Optimized trio genome sequencing (OTGS) as a first-tier genetic test in critically ill infants: practice in China. *Hum Genet*. 2020;139(4):473-482. doi:10.1007/s00439-019-02103-8
- 57. Kingsmore SF, Cakici JA, Clark MM, et al. A Randomized, Controlled Trial of the Analytic and Diagnostic Performance of Singleton and Trio, Rapid Genome and Exome Sequencing in III Infants. *Am J Hum Genet*. 2019;105(4):719-733. doi:10.1016/j. aihq.2019.08.009
- 58. Sanford EF, Clark MM, Farnaes L, et al. Rapid Whole Genome Sequencing Has Clinical Utility in Children in the PICU. Pediatr Crit Care Med. 2019;20(11):1007-1020. doi:10.1097/PCC.000000000000056
- 59. Petrikin JE, Cakici JA, Clark MM, et al. The NSIGHT1-randomized controlled trial: rapid whole-genome sequencing for accelerated etiologic diagnosis in critically ill infants. NPJ Genom Med. 2018;3:6. doi:10.1038/s41525-018-0045-8

- 60. Mestek-Boukhibar L, Clement E, Jones WD, et al. Rapid Paediatric Sequencing (RaPS): comprehensive real-life workflow for rapid diagnosis of critically ill children. *J Med Genet*. 2018;55(11):721-728. doi:10.1136/jmedgenet-2018-105396
- 61. Sun Y, Peng J, Liang D, et al. Genome sequencing demonstrates high diagnostic yield in children with undiagnosed global developmental delay/intellectual disability: A prospective study. *Hum Mutat*. 2022;43(5):568-581. doi:10.1002/humu.24347
- 62. French CE, Dolling H, Mégy K, et al. Refinements and considerations for trio whole-genome sequence analysis when investigating Mendelian diseases presenting in early childhood. *HGG Adv.* 2022;3(3):100113. doi:10.1016/j.xhqq.2022.100113
- 63. Stranneheim H, Lagerstedt-Robinson K, Magnusson M, et al. Integration of whole genome sequencing into a healthcare setting: high diagnostic rates across multiple clinical entities in 3219 rare disease patients. *Genome Med.* 2021;13(1):40. doi:10.1186/s13073-021-00855-5
- 64. Bertoli-Avella AM, Beetz C, Ameziane N, et al. Successful application of genome sequencing in a diagnostic setting: 1007 index cases from a clinically heterogeneous cohort. *Eur J Hum Genet*. 2021;29(1):141-153. doi:10.1038/s41431-020-00713-9
- 65. Riley LG, Cowley MJ, Gayevskiy V, et al. The diagnostic utility of genome sequencing in a pediatric cohort with suspected mitochondrial disease. *Genet Med.* 2020;22(7):1254-1261. doi:10.1038/s41436-020-0793-6
- 66. Vanderver A, Bernard G, Helman G, et al. Randomized Clinical Trial of First-Line Genome Sequencing in Pediatric White Matter Disorders. *Ann Neurol*. 2020;88(2):264-273. doi:10.1002/ana.25757
- 67. Scocchia A, Wigby KM, Masser-Frye D, et al. Clinical whole genome sequencing as a first-tier test at a resource-limited dysmorphology clinic in Mexico. NPJ Genom Med. 2019;4:5. doi:10.1038/s41525-018-0076-1
- 68. Thiffault I, Farrow E, Zellmer L, et al. Clinical genome sequencing in an unbiased pediatric cohort. *Genet Med.* 2019;21(2):303-310. doi:10.1038/s41436-018-0075-8
- 69. Beuschel J, Geyer H, Rich M, et al. Leveraging Rapid Genome Sequencing to Alter Care Plans for Pediatric Patients in a Community Hospital Setting in the United States. *J Pediatr*. Dec 2021;239:235-239. doi:10.1016/j.jpeds.2021.08.010
- 70. Willig LK, Petrikin JE, Smith LD, et al. Whole-genome sequencing for identification of Mendelian disorders in critically ill infants: a retrospective analysis of diagnostic and clinical findings. *Lancet Respir Med.* 2015;3(5):377-387. doi:10.1016/S2213-2600(15)00139-3
- 71. Lindstrand A, Ek M, Kvarnung M, et al. Genome sequencing is a sensitive first-line test to diagnose individuals with intellectual disability. *Genet Med.* 2022;24(11):2296-2307. doi:10.1016/j.gim.2022.07.022
- 72. Lee HF, Chi CS, Tsai CR. Diagnostic yield and treatment impact of whole-genome sequencing in paediatric neurological disorders. *Dev Med Child Neurol*. 2021;63(8):934-938. doi:10.1111/dmcn.14722
- 73. Bick D, Fraser PC, Gutzeit MF, et al. Successful Application of Whole Genome Sequencing in a Medical Genetics Clinic. *J Pediatr Genet*. 2017;6(2):61-76. doi:10.1055/s-0036-1593968
- 74. Bowling KM, Thompson ML, Amaral MD, et al. Genomic diagnosis for children with intellectual disability and/or developmental delay. *Genome Med*. 2017;9(1):43. doi:10.1186/s13073-017-0433-1
- 75. Gilissen C, Hehir-Kwa JY, Thung DT, et al. Genome sequencing identifies major causes of severe intellectual disability. *Nature*. 2014;511(7509):344-347. doi:10.1038/nature13394
- 76. Soden SE, Saunders CJ, Willig LK, et al. Effectiveness of exome and genome sequencing guided by acuity of illness for diagnosis of neurodevelopmental disorders. *Sci Transl Med*. 2014;6(265):265ra168. doi:10.1126/scitranslmed.3010076
- 77. Palmquist R, Jenkins SM, Bentley D, et al. Evaluating use of changing technologies for rapid next-generation sequencing in pediatrics. Pediatr Res. 2022;92(5):1364-1369. doi:10.1038/s41390-022-01965-5
- 78. Sweeney NM, Nahas SA, Chowdhury S, et al. Rapid whole genome sequencing impacts care and resource utilization in infants with congenital heart disease. NPJ Genom Med. 2021;6(1):29. doi:10.1038/s41525-021-00192-x
- 79. Dimmock DP, Clark MM, Gaughran M, et al. An RCT of Rapid Genomic Sequencing among Seriously III Infants Results in High Clinical Utility, Changes in Management, and Low Perceived Harm. *Am J Hum Genet*. 2020;107(5):942-952. doi:10.1016/j. ajhg.2020.10.003
- 80. van Diemen CC, Kerstjens-Frederikse WS, Bergman KA, et al. Rapid Targeted Genomics in Critically III Newborns. *Pediatrics*. 2017;140(4):e20162854. doi:10.1542/peds.2016-2854
- 81. Stutterd CA, Vanderver A, Lockhart PJ, et al. Unclassified white matter disorders: A diagnostic journey requiring close collaboration between clinical and laboratory services. *Eur J Med Genet*. 2022;65(9):104551. doi:10.1016/j.ejmg.2022.104551
- 82. van der Sanden BPGH, Schobers G, Corominas Galbany J, et al. The performance of genome sequencing as a first-tier test for neurodevelopmental disorders. Eur J Hum Genet. 2023;31(1):81-88. doi:10.1038/s41431-022-01185-9
- 83. Schon KR, Horvath R, Wei W, et al. Use of whole genome sequencing to determine genetic basis of suspected mitochondrial disorders: cohort study. *BMJ*. 2021;375:e066288. doi:10.1136/bmj-2021-066288
- 84. Owen MJ, Niemi AK, Dimmock DP, et al. Rapid Sequencing-Based Diagnosis of Thiamine Metabolism Dysfunction Syndrome. *N Engl J Med.* 2021;384(22):2159-2161. doi:10.1056/NEJMc2100365
- 85. Sawyer's Journey. facebook.com/sawyersjourneyTRIP12/. Created March 13, 2014. Accessed December 15, 2021.
- 86. Owen MJ, Lefebvre S, Hansen C, et al. An automated 13.5 hour system for scalable diagnosis and acute management guidance for genetic diseases. *Nat Commun.* 2022;13(1):4057. doi:10.1038/s41467-022-31446-6
- 87. Goranitis I, Wu Y, Lunke S, et al. Is faster better? An economic evaluation of rapid and ultra-rapid genomic testing in critically ill infants and children. *Genet Med.* 2022;24(5):1037-1044. doi:10.1016/j.gim.2022.01.013
- 88. Park A. Genetic testing is providing new hope for babies born with mysterious ailments. time.com/4951200/genetic-testing-providing-hope-babies-ailments/. TIME Magazine. Oct 2, 2017.
- 89. Illumina. Video: The story of Tree Baby. illumina.com/company/news-center/feature-articles/the-story-of-tree-baby.html. Published February 2020. Accessed January 6, 2023.
- 90. Willis M, Owen M, Lefebvre S, et al. eP286: Genome-to-treatment: A system to guide the acute management of genetic disorders in children. *Genetics in Medicine*. 2022.

- 91. Bick D. Bick SL. Dimmock DP, Fowler TA, Caulfield MJ, Scott RH, An online compendium of treatable genetic disorders, Am J Med Genet C Semin Med Genet. 2021;187(1):48-54. doi:10.1002/ajmg.c.31874
- 92. Kim J, Hu C, Moufawad El Achkar C, et al. Patient-Customized Oligonucleotide Therapy for a Rare Genetic Disease. N Engl J Med. 2019;381(17):1644-1652. doi:10.1056/NEJMoa1813279
- 93. Illumina. Baby Fitz's life-saving diagnosis through whole-genome sequencing, illumina.com/company/news-center/featurearticles/baby-fitz-life-saving-diagnosis-whole-genome-sequencing.html. Published April 2022. Accessed January 6, 2023.
- 94. Orphanet. Prevalence of rare diseases: Bibliographic data, Orphanet Report Series, Rare Diseases collection, January 2022, Number 1: Diseases listed in alphabetical order. orpha.net/orphacom/cahiers/docs/GB/Prevalence_of_rare_diseases_by_ alphabetical_list.pdf. Published Janary 2022. Accessed January 6, 2023.
- 95. ACMG Board of Directors. Points to consider in the clinical application of genomic sequencing. Genet Med. 2012;14(8):759-761. doi:10.1038/aim.2012.74
- 96. Binquet C, Leieune C, Faivre L, et al. Genome Sequencing for Genetics Diagnosis of Patients With Intellectual Disability: The DEFIDIAG Study. Front Genet. 2022;12:766964. doi:10.3389/fgene.2021.766964
- Jaitovich Groisman I, Hurlimann T, Shoham A, Godard B. Practices and views of neurologists regarding the use of whole-genome sequencing in clinical settings: a web-based survey. Eur J Hum Genet. 2017;25(7):801-808. doi:10.1038/ejhg.2017.64
- 98. Meng L, Pammi M, Saronwala A, et al. Use of Exome Sequencing for Infants in Intensive Care Units: Ascertainment of Severe Single-Gene Disorders and Effect on Medical Management. JAMA Pediatr. 2017;171(12):e173438. doi:10.1001/ jamapediatrics.2017.3438
- 99. Illumina. YouTube. Ending Sophia's 10-Year Diagnostic Odyssey. youtu.be/IDPWoWF4cqw. Accessed January 6, 2023.
- 100. Malinowski J, Miller DT, Demmer L, et al. Systematic evidence-based review: outcomes from exome and genome sequencing for pediatric patients with congenital anomalies or intellectual disability. Genet Med. 2020;22(6):986-1004. doi:10.1038/s41436-
- 101. Lavelle TA, Feng X, Keisler M, et al. Cost-effectiveness of exome and genome sequencing for children with rare and undiagnosed conditions. Genet Med. 2022;24(6):1349-1361. doi:10.1016/j.gim.2022.03.005
- 102. Chinese Medical Association Medical Genetics Branch. Expert consensus on the clinical application of whole-genome sequencing in the detection of genetic diseases. Chinese Journal of Pediatrics, 2019;57(6):419-423, doi:10.3760/cma.j.is sn.0578-1310.2019.06.004
- 103. Sachdev R, Field M, Baynam GS, et al. Paediatric genomic testing: Navigating medicare rebatable genomic testing. J Paediatr Child Health. 2021;57(4):477-483. doi:10.1111/jpc.15382
- 104. Boycott K, Hartley T, Adam S, et al. The clinical application of genome-wide sequencing for monogenic diseases in Canada: Position Statement of the Canadian College of Medical Geneticists. J Med Genet. 2015;52(7):431-437. doi:10.1136/ imedgenet-2015-103144
- 105. Project Baby Manatee. Nicklaus Children's Hospital. Advanced Genomics for Critically III Children Final Report. nicklauschildrens. org/NCH/media/docs/pdf/research/Final-report-State-Appropriations-NCH-PMI.PDF, Accessed January 6, 2023.
- 106. Rady Children's Institue Genomic Medicine. Expanding Access: State Projects. Michigan's Project Baby Deer, radygenomics.org/ case-studies/state-projects/. Published December 2021. Accessed January 6, 2023.
- 107. Franck LS, Scheurer-Monaghan A, Bupp CP, et al. Healthcare Professionals' Attitudes toward Rapid Whole Genome Sequencing in Pediatric Acute Care. Children (Basel). 2022;9(3):357. doi:10.3390/children9030357
- 108. Medica Magazine. News. Rare diseases: rapid diagnosis instead of a long odyssey, medica-tradefair.com/en/News/News_ from_the_Editors/Rare_diseases_rapid_diagnosis_instead_of_a_long_odyssey. Published April 26, 2022. Accessed January 6,
- 109. Illumina. Press release. Illumina Supports Leading German Hospital to Accelerate Genetic Disease Diagnosis in Critically III Children. illumina.com/company/news-center/press-releases/2022/98b8f63c-7e56-4e64-b669-6a8e7a5e3efc.html. Published April 25, 2022, Accessed January 6, 2023,
- 110. Illumina. Press release. Illumina Supports Israel's National Program to Accelerate Diagnoses of Critically-Ill Newborns with Suspected Genetic Disease. illumina.com/company/news-center/press-releases/2021/b4a26ac9-6d83-4d0a-98c7-3f1962b1a7c4.html. Published October 7, 2021. Accessed January 6, 2023.
- 111. Cortial L, Nguyen C, Julkowska D, et al. Managing rare diseases: examples of national approaches in Europe, North America and East Asia. Rare Dis Orphan Drugs J. 2022;1:10. 10.20517/rdodj.2022.02
- 112. PRECISE Precision Health Research Singapore. npm.sq/. Accessed January 6, 2023.
- 113. WINGS Wales Infants & Children's Genome Service. All Wales Medical Genomics Service. medicalgenomicswales.co.uk/index. php/health-professional-information/wings. Published September 27, 2019. Accessed January 6, 2023.
- 114. NHS England. National Genomic Test Directory. england.nhs.uk/publication/national-genomic-test-directories/. Updated October 31, 2022. Accessed January 6, 2023.
- 115. Blue Shield of California. Press release. Blue Shield of California become first health plan in U.S. to cover cost of rapid whole genome sequencing for critically ill children. news.blueshieldca.com/2020/03/09/RADY-genomics. Published March 9, 2020. Accessed January 6, 2023.
- 116. Takahashi Y, Date H, Oi H, et al. Six years' accomplishment of the Initiative on Rare and Undiagnosed Diseases: nationwide project in Japan to discover causes, mechanisms, and cures. J Hum Genet. 2022;10.1038/s10038-022-01025-0. doi:10.1038/ s10038-022-01025-0
- 117. BioPharmaAPAC. Illumina reaffirms commitment to the National Project of Bio Big Data in Korea. biopharmaapac.com/ news/83/749/illumina-reaffirms-commitment-to-the-national-project-of-bio-big-data-in-korea.html. Published September 20, 2021. Accessed January 6, 2023.

- 118. Illumina. Press release. Illumina and China's Children's Hospital of Fudan Bring Rapid WGS to the NICU. illumina.com/company/ news-center/feature-articles/illumina-and-children-s-hospital-of-fudan-in-china.html. Published April 22, 2019. Accessed January 6, 2023.
- 119. Australian Government Medical Services Advisory Committee (MSAC). Public Summary Document. Genetic testing for childhood syndromes. msac.gov.au/internet/msac/publishing.nsf/Content/7263C1A2C7F06220CA2580EA0007A3F3/\$File/1476%20-%20 Final%20PSD-updated%20with%20Aug%202019%20outcome.pdf. Publised August 2019. Accessed January 6, 2023.
- 120. Halabi N, Ramaswamy S, El Naofal M, et al. Rapid whole genome sequencing of critically ill pediatric patients from genetically underrepresented populations. *Genome Med.* 2022;14(1):56. doi:10.1186/s13073-022-01061-7
- 121. Illumina. How one children's hospital is transforming care for kids in Dubai. illumina.com/company/news-center/feature-articles/how-one-children-s-hospital-is-transforming-care-for-kids-in-dub.html. Published July 26, 2022. Accessed May 2, 2023.
- 122. Marshall CR, Bick D, Belmont JW, et al. The Medical Genome Initiative: moving whole-genome sequencing for rare disease diagnosis to the clinic. *Genome Med.* 2020;12(1):48. doi:10.1186/s13073-020-00748-z
- 123. Illumina. Press release. Illumina and Genetic Alliance Launch \$120 Million Global Initiative to Increase Equity and Improve Outcomes for Families Impacted by Genetic Disease. illumina.com/company/news-center/press-releases/2021/30ac0427-f1cb-4088-8e12-1df6f4279ffa.html. Published November 16, 2021. Accessed January 6, 2023.
- 124. iHope Genetic Health website. Enabling clinical whole genome sequencing for undiagnosed individuals across the globe. ihopegenetichealth.org/. Accessed January 6, 2023.
- 125. Cohn I, Paton TA, Marshall CR, et al. Genome sequencing as a platform for pharmacogenetic genotyping: a pediatric cohort study. NPJ Genom Med. 2017;2:19. doi:10.1038/s41525-017-0021-8
- 126. Schobers G, Schieving JH, Yntema HG, et al. Reanalysis of exome negative patients with rare disease: a pragmatic workflow for diagnostic applications. *Genome Med.* 2022;14(1):66. doi:10.1186/s13073-022-01069-z
- 127. Arth AC, Tinker SC, Simeone RM, Ailes EC, Cragan JD, Grosse SD. Inpatient Hospitalization Costs Associated with Birth Defects Among Persons of All Ages United States, 2013. MMWR Morb Mortal Wkly Rep. 2017;66(2):41-46. doi:10.15585/mmwr. mm6602a1
- 128. Global Disease Commission to end the diagnostic odyssey for children with a rare disease. globalrarediseasecommission.com/ Report. Published 2019. Accessed January 6, 2023.



1.800.809.4566 toll-free (US) | +1.858.202.4566 tel

techsupport@illumina.com | www.illumina.com